

## Utilization of a Next Generation Decellularized Dermal Allograft to Augment the Repair of a Massive Rotator Cuff Tear Involving the Supraspinatus and Infrapinatus Tendons

Robert F. Hines, MD  
Oklahoma City, OK  
Board Certified, American Board of Orthopaedic Surgeons

### Surgical Summary

#### Clinical Presentation:

- 44 year-old male, healthy police/SWAT officer who slipped and fell on his elbow, dislocating his shoulder. Shoulder was reduced, and patient underwent rehab for six weeks, but made no progress.
- Exam revealed 90 degrees of active forward flexion but full passive range of motion.
- Manual muscle testing: 3+/5 supraspinatus and infrapinatus. No clinical instability.
- MRI revealed 4 – 5 cm tears in the supraspinatus and infrapinatus tendons, no muscle atrophy and no labral tears.

#### Intraoperative Findings:

- Massive rotator cuff tear, involving the supraspinatus and infrapinatus tendons, retracted to the mid humeral head (Figure 1 and 2).

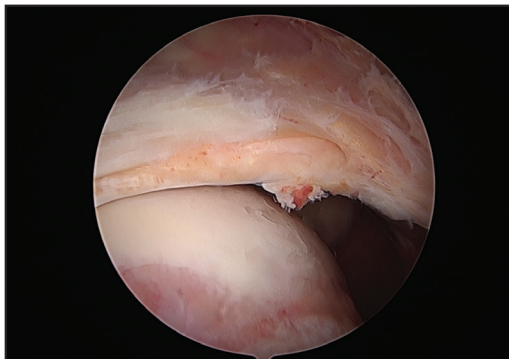


Figure 1

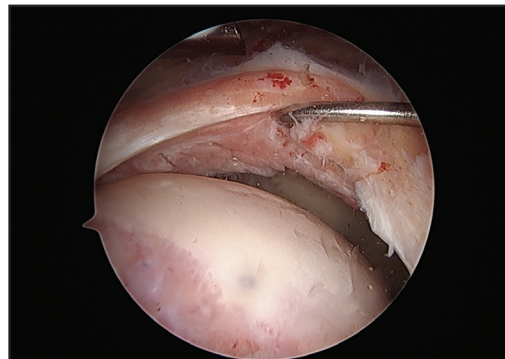


Figure 2

#### Surgeon Perspective:

*“DermaPure® added augmentation to this massive rotator cuff repair. At 3 months post-op, this patient continues with the 10 pound lifting restriction and remains pain free.”*

*All of the patients in which I have utilized DermaPure® for augmentation of the rotator cuff repair, no longer required narcotics for pain management by post-op day 4 – 7.”*

## Utilization of a Next Generation Decellularized Dermal Allograft to Augment the Repair of a Massive Rotator Cuff Tear Involving the Supraspinatus and Infrapinatus Tendons

Robert F. Hines, MD  
Oklahoma City, OK  
Board Certified, American Board of Orthopaedic Surgeons

### Surgical Procedure:

- Right shoulder arthroscopy with subacromial decompression, distal clavicle excision and deltoid splitting incision.
- Supraspinatus and infrapinatus tendons were mobile and easily advanced to the footprint. Double row rotator cuff repair of the supraspinatus and infrapinatus rotator cuff tendons using 4.5 mm Healix BR™ anchors medially, passing Permacord® and Permatape® 5.0 mm laterally to the musculotendon junction. Medial Permacord® tied, then Permatape® passed through DermaPure® decellularized dermal allograft, previously soaked in bacitracin/vancomycin solution.
- Repair completed by placing DermaPure® decellularized dermal allograft over the repair, using a suture bridge, then completed double row repair with Healix BR™ anchors laterally to fit the graft without tension (Figure 3).

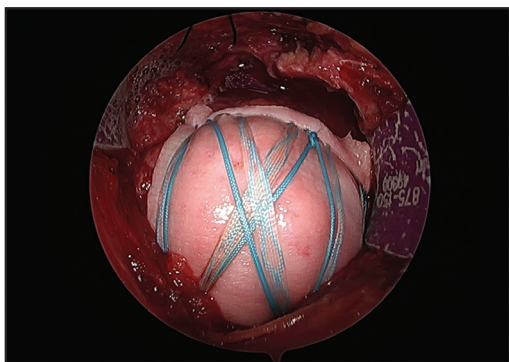


Figure 3

#### DermaPure® Implantation:

- 4 cm x 6 cm
- Basement membrane outer-most

### Post-Operative Note:

- By post op day 4, the patient no longer required narcotics for pain management.
- Patient is being followed routinely to assess clinical outcomes.

TissueRegenixUS.com  
1-855-452-0133  
1808 Universal City Blvd, Universal City, TX 78148

Healix BR™ is a trademark of DePuy Mitek Inc., a Johnson and Johnson Company.  
PermaCord® is a registered trademark of Johnson and Johnson.  
PermaTape® is a registered trademark of DePuy Synthes, Inc., a Johnson and Johnson Company.

DermaPure® is a registered trademark of Tissue Regenix Wound Care, Inc. dCELL® Technology is a registered trademark of Tissue Regenix Limited. TR040